# Turny® 6-Way

SV Bruksanvisning

EN User manual

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SV

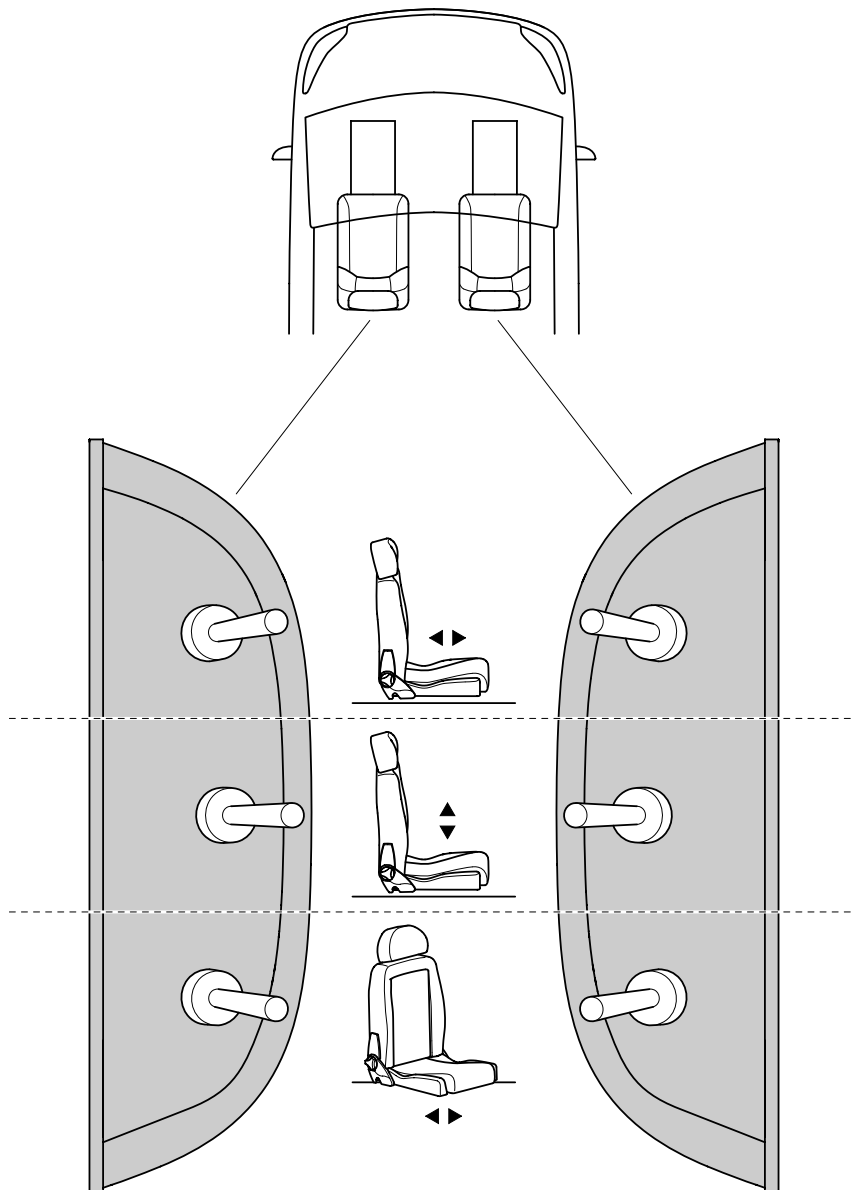
EN

DE

FR

ES

IT

SV

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-  *Weight capacity max 180 kg.*
-  *The Forward/Backwards and Up/Down toggle switches are always on. Ensure that they are not accidentally activated while driving. The Left/Right toggle switch is off when the vehicle's ignition is turned on.*
-  *Do not drive with Turny® 6-Way adjusted more than 240 mm backwards (Comfort adjustment).*
-  *Using the Turny® 6-Way without the optional footrest causes a potential pinch risk.*
-  *Any modifications, adjustments or repairs to a Turny® 6-Way must be performed by an authorised technician. Always contact your dealer for disassembly.*
-  *The control unit and the toggle switch cover shall not be used as support when transferring to and from the seat.*
-  *When jump starting the vehicle, temporarily remove the product's fuse.*

**Product data**

Length*	1150 mm
Width	430 mm
Height	170-347 mm
Comfort adjustment	704 mm
Vertical travel	177 mm
Swivel range**	0-140° / 0-120°
Weight capacity***	180 kg
Unit weight, without seat and accessories..	54 kg
Operating voltage	12V
Fuse****	20A

\* can be reduced 45-190 mm by installer if needed.  
 \*\* differs between left (140°) and right (120°) side installation.  
 \*\*\* refers to the total weight of the seat, its accessories and the user.  
 \*\*\*\* when replacing, use a 20 A fuse.

**Care instructions**

As necessary.....	Carefully wipe off and vacuum clean the Turny® 6-Way. Do not use any liquids.
Once every two years.....	We recommend to contact your dealer for an overhaul.

**Troubleshooting**

Turny® 6-Way is not working .....	Check that / - the fuse has not blown and that the vehicle´s battery is charged up. - there is no obstruction blocking the selected movement. If the problem remains, contact your dealer.
Forward/backward movement... is working slower than usually.	Probably time for re-lubrication of the aluminum profiles. Contact BraunAbility Support for further help.
Left/right toggle switch is..... working despite the vehicle's ignition is turned on.	Contact your dealer.
Discordant sounds are heard..... during operation.	Contact your dealer.

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